ENGLISH: How to Configure Language on MolecuLight i:X Imaging Device

STEP 1	STEP 2	STEP 3	STEP 4	STEP 5
<p>Power on the MolecuLight i:X Display Screen by holding the Display Screen Power Button.</p> <p>If already powered on, press the Home Button to illuminate the Display Screen.</p> 	<p>“slide to unlock” to unlock the Display Screen.</p> 	<p>Swipe to the center screen to access the “Settings”.</p> 	<ol style="list-style-type: none"> Under Settings select “General” Under General select “Language & Region” Under Language & Region select “iPod Language” Under iPod Language, select the desired language Select “Done” to confirm the language Select “Change to...” to proceed with language change 	<p>NOTE: Step 5 must be completed to finish the language configuration. Do not skip Step 5.</p> <p>To complete the language change, power the Display Screen OFF and then ON as follows:</p> <ol style="list-style-type: none"> Hold the Display Screen Power Button and when prompted, “slide to power off” to power off the Display Screen. Hold the Display Screen Power Button again until the Display Screen powers on. <p>The MolecuLight i:X is now configured to the selected language.</p>

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